

# AmnioTest™

Rapid Amniotic fluid swab test





The AmnioTest™ is designed to assist in the detection of rupture of the fetal membranes, based on elevation of the pH in the upper vagina consistent with leakage of amniotic fluid. The sterile swab impregnated with nitrazine yellow dye, offers a rapid, reliable, cost effective and simple to use point-of-care test to aid in the detection of amniotic fluid in patients with suspected PROM.



- \* Sterile for patient safety
- \* Rapid, safe and easy to use
- \* Test results in 15 seconds
- \* No speculum required
- \* Easy peel opening
- \* Up to 3-years shelf life
- \* Canadian manufactured product
- \* Easy to QC with available buffers to match colour interpretation card
- \* Education PowerPoint available for staff training to meet POC requirements

#### Contact us:

 Toll Free: 800-268- 2341  
Tel: 905-731-0300

 [support@pro-lab.com](mailto:support@pro-lab.com)

# AmnioTest™

Rapid Amniotic fluid swab test




1 Peel



Remove swab from protective sleeve (keep swab tip dry away from any liquid)

2 Collect Sample

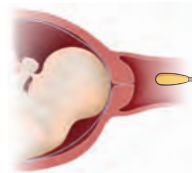

15 sec.

Insert swab into vagina ensuring that nitrazine tip gently comes in contact with the upper cervical area for about 15 seconds

3 Interpret result





Carefully remove swab from vagina and immediately examine colour with AmnioTest™ colour card



Enter catalogue # at [www.pro-lab.com](http://www.pro-lab.com) for Instructions For Use

## Ordering Information

To place your order, please contact us at:

 1-800-268-2341  
 support@Pro-Lab.com

Product Code	Description	Size
PL.901	AmnioTest™	100 Swabs
PL. 901-20	AmnioTest™	20 Swabs
PL.902	AmnioTest™ Colour Interpretation Cards	50 Cards
PL.910	Buffer Solution pH 5.0	2.0 ml
PL.911	Buffer Solution pH 5.5	2.0 ml
PL.912	Buffer Solution pH 6.0	2.0 ml
PL.913	Buffer Solution pH 6.5	2.0 ml
PL.914	Buffer Solution pH 7.0	2.0 ml
PL.915	Buffer Solution pH 7.5	2.0 ml